

ARTWORK PANEL



ALLIANCE

Master Ref:

Sl602597

Trident Reference No:

TR3136558

SKU Description:

Ashton & Parsons Teething Gel
10ml UK LFT

Brand:

Alliance

Range:

Alocclair

Product:

Teething Gel

Pack Type:

Leaflet

Pack Size:

10 ml

Action:

C

Date:

30/05/2024

Country:

UK

CAD Ref:

BK-LFT-150x210mm

Manufacturing Site:

COC

Printer:

Grafiche Thiella srl (Milan, Italy)

Substrate:

Propharma 50g

Barcode Type:

N/A

Magnification:

N/A

Pharmacode No/NE:

N/A

Technical & Non Printing Items:

Dieline

Guides

Colours

PANTONE 280 C

1

LTH

PANTONE 1775 C

2

LTH



Connaught House, Connaught Road,
Kingswood Business Park, Hull, HU7 3AP, England.
T: +44 (0) 1482 828100

Please note that any low resolution paper Canon colour
copies associated with this job should be referred to for
content, layout and colour separation only.

UNDER NO CIRCUMSTANCES SHOULD THIS
ARTWORK BE ALTERED WITHOUT PRIOR
PERMISSION FROM TRIDENT.

CUSTOMER INFO

Minimum Point Size = 3.20pt

150mm

210mm

26,25mm



Ashton & Parsons®
TEETHING GEL
For teething pain relief

Ashton & Parsons® Teething Gel
Information leaflet

Intended use:

Ashton & Parsons® Teething Gel is intended to be used in the oral cavity for relieving the discomfort and pain associated with teething: eruption of a tooth.

Ashton & Parsons® Teething Gel rapidly forms an invisible film which helps to protect gums from the pain and discomfort associated with teething. During use, the applicator, equipped with silicone bristles, gently massages the gums.


Ashton & Parsons® Teething Gel has a pleasant banana flavour.


Ashton & Parsons® Teething Gel does not contain lactose, sugar, alcohol or local anaesthetics.




Instructions for use:

Apply a pea sized amount of the product to the affected area and gently massage gums with the applicator. In infants, apply preferably after meals and before baby's sleep to promote film build up and prolong pain relief. Use 3-4 times a day as needed. Return cap to the top of the applicator after each application.
Ashton & Parsons® Teething Gel is suitable for use in children over the age of 3 months.

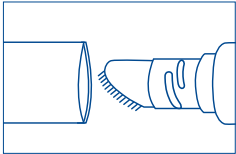


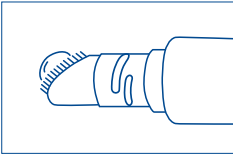




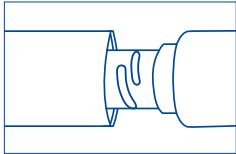


Before using the product remove any dried residual with clean dry hands

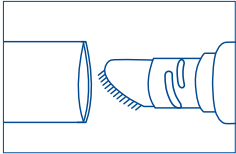

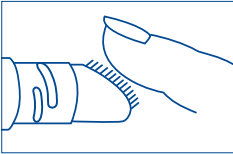



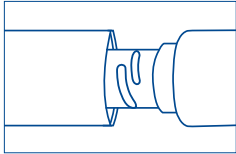

ARTWORK PANEL

ALLIANCE

Master Ref:

SI602597

Trident Reference No:

TR3136558

SKU Description:

Ashton & Parsons Teething Gel
10ml UK LFT

Brand:

Alliance

Range:

Alocclair

Product:

Teething Gel

Pack Type:

Leaflet

Pack Size:

10 ml

Action:

C

Date:

30/05/2024

Country:

UK

CAD Ref:

BK-LFT-150x210mm

Manufacturing Site:

COC

Printer:

Grafiche Thiella srl (Milan, Italy)

Substrate:

Propharma 50g

Barcode Type:

N/A

Magnification:

N/A

Pharmacode No/NE:

N/A

Technical & Non Printing Items:

Dieline

Guides

Colours

PANTONE 280 C

1

LTH

PANTONE 1775 C

2

LTH

TRIDENT

Connaught House, Connaught Road,
Kingswood Business Park, Hull, HU7 3AP, England.
T: +44 (0) 1482 828100

Please note that any low resolution paper Canon colour
copies associated with this job should be referred to for
content, layout and colour separation only.

UNDER NO CIRCUMSTANCES SHOULD THIS
ARTWORK BE ALTERED WITHOUT PRIOR
PERMISSION FROM TRIDENT.

CUSTOMER INFO

Minimum Point Size = 3.20pt

150mm

210mm

26,25mm

All babies teethe at different rates, but this is a general guide of what to expect.

Upper Jaw

Lower Jaw

Which Tooth Next?

A

B

C

F

E

G

G

E

F

D

A

D

F

E

G

A

5 – 7 months

Bottom front teeth (incisors) usually emerge first

B

6 – 8 months

Top front teeth (insicors) emerge

C

9 – 11 months

Top lateral incisors emerge, either side of the top front teeth

D

10 – 12 months

Bottom lateral incisors emerge, either side of the bottom front teeth

E

12 – 16 months

Molars (back teeth) emerge

F

16 – 20 months

Canines emerge

G

20 – 30 months

Second molars (back teeth) emerge

Ingredients:

Aqua, PVP (12%), maltodextrin, propylene glycol, polyethylene glycol, hydrogenated castor oil, xanthan gum, banana flavour, potassium sorbate, sodium benzoate, sodium hyaluronate (0.1%), xylitol, disodium EDTA, cetylpyridinium chloride.

Warnings and precautions:

Each Ashton & Parsons® Teething Gel tube should be used for a single patient to avoid cross contamination of the applicator and gel. The use of Ashton & Parsons® Teething Gel is contraindicated for patients with a history of hypersensitivity to any of the ingredients. Do not use if you're pregnant, think you're pregnant, or are lactating. Keep out of reach of children.

Contents: 10ml tube.

The product should only be applied by an adult.

This medical device is not a toy. **NEVER** leave the medical device in the hands of a child to avoid risk of choking. Check the medical device before use for any signs on wear and tear and pay attention during the application of the gel that there is no accidental detachment of any of the components parts of the applicator head. In case, discontinue use. Do not try to reassemble the detached part. Do not use if the tube is open or damaged.

Any incident experienced whilst using this product should be reported to the manufacturer:
pharmacovigilance@alliancepharma.co.uk.

Serious deterioration of health that has occured in relation to this product should also be reported to the competent authority of your country.

Dispose of responsibly and in accordance with local regulations.

Store between 5°C and 25°C.

Stop using the device once the gel has run out.

Do not use after expiry date.

Rev 1: 2024-03

UK
CA

i

1

5 °C

25 °C

MD

Ashton & Parsons, Alliance and associated devices are
registered trademarks of Alliance Pharmaceuticals Limited
© Alliance Pharmaceuticals Limited 2024

Alliance Pharmaceuticals Ltd,
Avonbridge House, Bath Road, Chippenham,
Wiltshire SN15 2BB, United Kingdom

020551

TR3136558